## • **Welcome**

Congratulations on your new iHealth Wireless Body Analysis Scale with Wi-Fi 802.11 b/g/n technology. This Scale tracks your weight, BMI and your body composition, and along with the free companion “iHealth MyVitals” app, can help you stay motivated and reach your weight goals.

## • **Mobile Device Compatibility**

The iHealth Wireless Body Analysis Scale works with the following devices:  
Apple devices using iOS system version 6.0 and higher:

- iPhone 4 or later
- iPad mini or later
- iPad (2rd generation) or later
- iPod touch (4th generation) or later

Android devices using system version 4.0 and higher:

- Samsung® GALAXY S2, S3, S4, S5
- Samsung® NOTE II, III
- Samsung® Mega 6.3
- LG® NEXUS 4, 5
- HTC One® M7, M8
- HTC Droid DNA

Compatible devices are subject to change. For the latest compatibility list, go to [www.ihealthlabs.com/support](http://www.ihealthlabs.com/support)

## • **Wi-Fi Connectivity**

The iHealth Wireless Body Analysis Scale is compatible with home routers using standard Wi-Fi 802.11 b/g/n (2.4Ghz), WEP, WPA and WPA2 personal security.

## • **Download the Free iHealth App**


Scan the QR code and download the free “iHealth MyVitals” companion app or download directly from the Apple App Store or Google Play. It is very important that each user follows the on-screen instructions to register and create their personal profile with height and age in order to accurately calculate body composition measurements.

▶ **Press The "SET" Button**

Make sure the scale is setup ready.

The scale will display "SET" the first time you power it on. If you do not see "SET" display on your scale, press the Set button under the battery cover.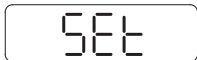

▶ **Set Up The WI-FI Connection With Scale (Prior to first use, or when changing routers.)**

STEP 1: Remove the tab inserted in the battery tray.

STEP 2: Connect your mobile device to home Wi-Fi.

STEP 3: Make sure your Scale displays "SET". Launch the iHealth MyVitals App. Select your iHealth Wireless Body Analysis Scale and follow the on screen instruction to set up.

STEP 4: Scan the QR Code on the back of Scale to link it to your account. Follow on-screen instruction to enter the password for your Router. When the Wi-Fi connection is being established, the wireless symbol on your Scale will start blinking and then stabilize when the Wi-Fi connection is successful. Your scale is now connected to your home Wi-Fi and ready to use.



▶ **Measuring Your Body Composition Index**


a. Launch the iHealth App. Tap the Scale to turn it on, wait until "0.0" displays and the Wi-Fi symbol stabilizes.

b. Stand on all four electrodes with bare feet. Remain on the scale for a few seconds to allow the scale to continue the measurement. Your weight will appear on the Scale, followed by Body Fat. If you have socks on, the Scale will only display your weight.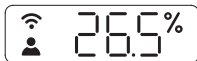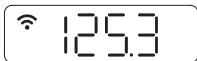



You can view your detailed body composition data on the iHealth App.  
Note: If the Body Fat measurement fails, only your weight will be displayed.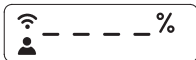

#### • **Measuring Room Temperature and Humidity**


To check your room temperature and humidity.

1. Tap on the scale to wake it up. You will see 0.0 light up on the display.
2. Tap on the right or left bottom corner of the scale, room temperature and humidity will display for a few seconds.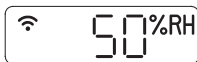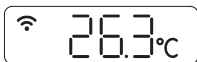

#### • **Taking Measurements Without a Mobile Device**

The iHealth Wireless Body Analysis Scale allows up to 10 users. When there are multiple users, the Scale determines who the user is by matching the new weight with the weight previously recorded. If the weight of two or more users is similar, the Scale will display the user number (e.g. User # "x"). Step lightly on the bottom left or right corner to select the appropriate user, and the scale will confirm it after three seconds. Your measurement data will be uploaded to the Cloud automatically.

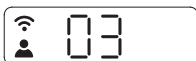## ▸ **SOFTWARE UPDATES**

From time to time, the Scale's firmware is updated to enhance performance. When a firmware update is available, a notification message will appear in the app. Complete the following steps to install the update:

- a. Follow the steps in the " SET UP WI-FI CONNECTION WITH SCALE " section.
- b. You will see a notification message in the app asking for permission to proceed. Select "Yes".
- c. The update will begin to download. You will see a cursor display scale moving from left to right. The scale will display "0.0" when the update is complete.

## ▸ **Instructions for switching ISP or wireless router**

If you switch your ISP or internet router, press the "Set" button under the battery cover on the back of the Scale to reset the Wi-Fi connection, then follow the "SET UP THE SCALE'S WI-FI CONNECTION" instruction to reconnect the Scale. Visit <http://www.ihealthlabs.com> (for US) or <http://www.ihealthlabs.eu> (for EU) for additional information.

For Customer Service, please call +1(855) 816-7705 (USA) or +33(0) 1 44 94 04 81 (EU).

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